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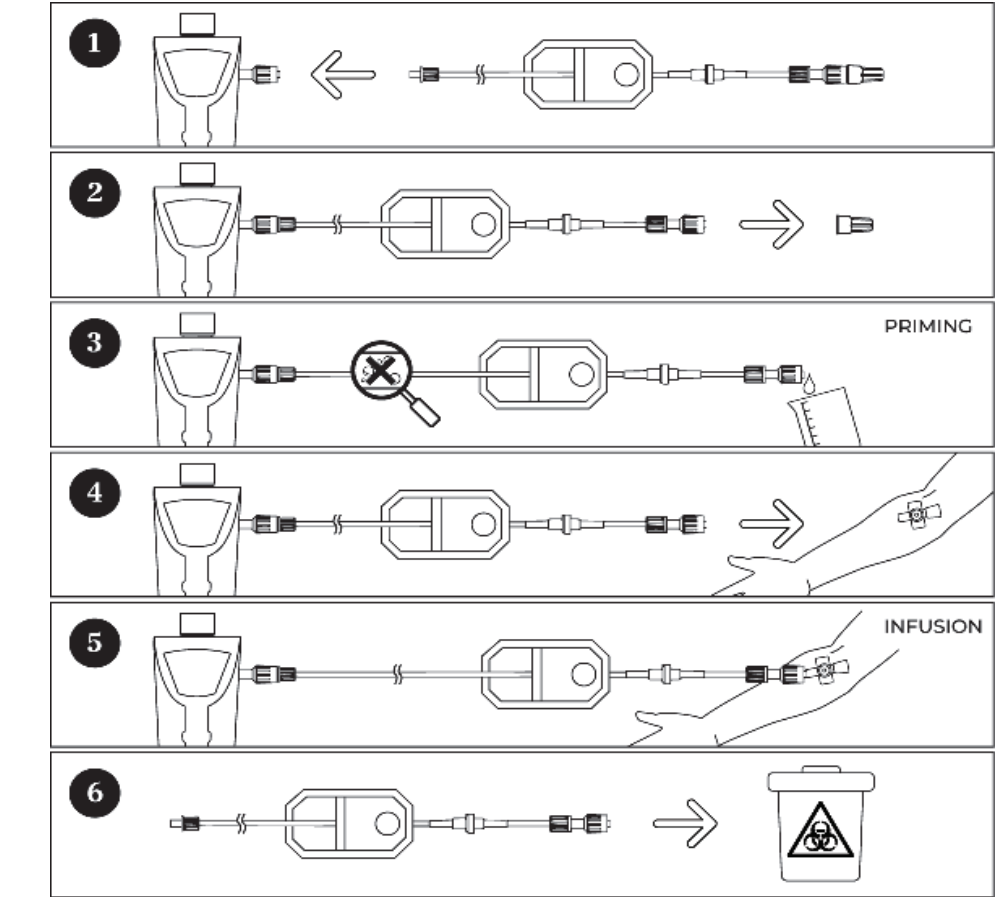
EN

O15 H2O PATIENT KIT AND O15 H2O PATIENT KIT EXTENDED

DESCRIPTION
Single use O15 H2O Patient Kits individually packaged per market unit. The device consist of tubing, a single way check valve, a 0.2 micron vented sterile filter and Luer connectors. The product is supplied sterile and pre-assembled. The device is to be used only with the P3 MT-100.

CONTENTS
REF MT-PAT-001, O15 H2O Patient Kits, 20 units
OR
REF MT-PAT-002, O15 H2O Patient Kit extended, 20 units.

INTENDED USE
Guiding a bolus of ¹⁵O H₂O water for injection and the saline flush into the patient venflon.

INTENDED USERS
Trained healthcare professionals working in the field of nuclear medicine, trained in aseptic techniques.

PATIENT TARGET GROUP
Adult patients undergoing ¹⁵O H₂O PET analysis.

STORAGE CONDITIONS
Store the device dry at room temperature, away from sunlight.

INCIDENT REPORTING
Any serious incident that has occured in relation to the device should be reported to the manufacturer and the competent authority.

PERFORMANCE CHARACTERISTICS
Max pressure: 3 bar
Sterilization by filtration through 0.2 micron sterile filter
Reverse flow prevented by single check valve

WARNINGS

- Do not reuse the Patient Kit, as it is single-use only. Resterilization or reuse of this device jeopardizes sterility and can lead to product dysfunction and cross-infection.
- Do not perform infusion without having primed the Patient Kit.
- Do not expose the ends of the Patient Kit to potential contamination.
- Do not use the Patient Kit if the sterile packaging is damaged or unintentionally opened before use.
- Do not use the Patient Kit if the label is unreadable or missing.
- Do not use the Patient Kit with other equipment than the Infusion kit (MT-INF-001), O15 H2O Production Kit (MT-PRO-001), and the P3 MT-100.
- Do not use anything other than Luer Locks in compliance with ISO 80369-7 to connect the Patient Kit to the patient's venflon.
- Used Patient Kits are bio-hazardous waste and should be handled accordingly. If assistance is needed for proper disposal of the device, please contact MedTrace.

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